

5 | SPECIALS

We offer a range of Special Products to help enhance your clients daily living in addition to their custom orthopedic shoes. These range from wet shoes for additional safety when bathing, for areas where the floor may be wet, or swimming right through to work shoes for additional safety when working or performing more hazardous activities. To complement the complete shoes, we also offer house shoes with some orthopedic adjustments, for safer indoor ambulation. Inner shoes can be ordered, which give the orthopedic support in a comfortable stand-alone device that the client can wear in different shoes. You will also find our range of test uppers which can be used to test the fit in a dynamic setting, allowing them to walk before committing to the complete shoe.

5.1 | HOW TO ORDER

Specials can be ordered through our regular upper and bottom order forms. Please indicate the special product code instead of the Model number.

MODEL		FILL OUT IN THE ORDER
1	Article	Article Number
2	Upper	Upper Height Use of material and colour Type of closure
3	Instep point	If you want to adjust our 1/3th standard, please fill out in cm to the front or to the back
4	Orthopedic options	Most orthopedic options can be made in our Special Products
5	Heel Height	Please fill out the heel height
6	Inlays	Please fill out the inlay order form if you do not supply your own inlays

5.2 | HOUSE SHOES

A pair of house shoes can be an ideal way to complement your clients orthopedic shoes, allowing them to have all the orthopedic modifications needed, in a soft comfortable material for indoor ambulation.



SP731



SP732

TECHNICAL SPECIFICATIONS

- Default:** A heel counter can be placed between the two layers of the backside
- Specials:** It is possible to order a specific reinforcement (BLU), see page 36 - 37
- Heel:** EVA wedge heel with out sole: no. 200 / 6 mm

House Shoes are available in several colours, please check our special material bundle no.; 1021-1036. Velcro can be ordered in several leather types.

5.3 | BATH- RUBBER AND SOCK SHOES

The bath shoe is worn during showering or bathing and the user can wear the heavier design of this shoe to go swimming (in chlorine, freshwater and salt water). Please note: this is not a hiking or walking shoe.

BATH SHOE



SP700

Lining: 3 mm ercoflex no: 5050
Upper material external: 4 mm thermoplastic EVA no: 5038

TECHNICAL SPECIFICATIONS

Default: This product can only be ordered as 'high' shoe and is only available in velcro closure
All orthopedic features are possible in the bath shoe

Specials:

- Suitable for chlorine- and salt water
- The wedge heel can, if required, be weighted with lead to enable swimming

Heel: Add-on heel; EVA with Nopair outer sole no.: 200 / 6 mm

RUBBER SHOE



SP710

TECHNICAL SPECIFICATIONS

Default:

Only high shoes and always with watertight tongue.
Available in Brown and Black

Specials:

- All orthopedic options can be ordered in Rubber Shoes
- Can be supplied with a steel nose
- These shoes are 100% watertight

Heel:

EVA heel with out sole: no. 200 / 6 mm

Lining:

3 mm ercoflex no: 5050

Upper material external:

2 mm crepina soft

SOCK SHOE



SP721



SP722

TECHNICAL SPECIFICATIONS

Default:

Suitable for all types of leather. Sock Shoes are being delivered without inner soles

Specials:

If specified, Sock Shoes can be delivered with anti-slip sole on the bottom and/or including inlays

Closure:

Sock Shoes can be ordered with any closure types

5.4 | WORK SHOES



W8001



W8002



W8007

SPECIFICATIONS

Standard:

The bottom part of work shoes are made with pre fabricated soles and have extra grip profile soles. All orthopedic options can be ordered in work shoes.

Options:

- Water tongue
- Gorex, between lining and leather
- Steel toe
- Steel sole plate
- Anti Penetration Insole / 5056
- Polyprop toe / carbon toe
- Rubber toe protection
- Rubber mudguard
- Oil and acid resistant sole nr.: 550, 605, 596

5.5 | TEST UPPERS AND INNER SHOES

TEST UPPERS



SP750

Size:

1	34-35	5	42-43
2	36-37	6	44-45
3	38-39	7	46-47
4	40-41		



SP751

Size:

18 cm		25 cm	
1	35-36-37	1	35-36-37
2	38-39-40	2	38-39-40
3	41-42-43	3	41-42-43
4	44-45-46	4	44-45-46



SP752

Size:

18 cm		25 cm	
1	35-36-37	1	35-36-37
2	38-39-40	2	38-39-40
3	41-42-43	3	41-42-43
4	44-45-46	4	44-45-46



SP753

Size:

18 cm			25 cm			40 cm		
1	35-36-37		1	35-36-37		1	35-36-37	
2	38-39-40		2	38-39-40		2	38-39-40	
3	41-42-43		3	41-42-43		3	41-42-43	
4	44-45-46		4	44-45-46		4	44-45-46	

INNER SHOE

The inner shoe can be produced with all orthopedic options and they can be worn in different shoes; as well orthopedic shoes as comfort shoes. Tongue runs from medial to lateral.



SP725 Inner Shoe with lacing



SP726 Inner Shoe with velcro

5.6 | DIABETIC LOOXZ MODELS

Our diabetic shoes are based on the design from the Looxz collection. The diabetic models have been modified in their construction and materials that make these models suitable for diabetics or those with sensitive feet.

SPECIFICATIONS

1. Technical information

The diabetic models are built the same as default in the construction and materials below:

PARTS OF SHOE	STANDARD DIABETIC SHAPE/CONSTRUCTION	STANDARD DIABETIC MATERIAL
Toe	Wing toe	Tep
Counter	Standard model	1.2 mm Renoflex for Men and Women
Complete Lining Oxford model	Front and tongue lining made in 1 piece	Choice 1011-1015 color dependent on design
Complete Lining Derby model	Front and tongue lining made in 1 piece Closing point not through lining	Choice 1011-1015 color dependent on design
Tongue	Not-seamless (standard) tongue with 3 mm padding	Choice 1011-1015 color dependent on design
Heel rounding	Standard	
Velcro	With roller buckle	
Sides	No side lining reinforcement	
Bottom	Wedge heel flat	55 Shore EVA build up material
Outer sole		Mandorlo- Vibram original or if needed CYS colors
Sportsband		EVA sportsband

2. Ordering

The diabetic models can be entered / selected under their specific model number in the CYStore. When using another order form, the model number must be entered. If no changes are required in the construction or materials, we will produce the shoes with the above specifications.

If necessary, the specifications can be adjusted per part of the shoe. Modify what is needed in the CYStore or state them in the order form. Only these items will be adjusted.

The standard prices and rules apply to other functional adjustments.

If design adjustments are made (adjustment of the upper leather, different type of bottom construction, etc. *) costs for "Off-Standard" will be charged.

3. Website

All diabetic models can be found at www.cysgroup.eu. By means of the filter option, models for men / women can be selected.

** Applicable rules Looxz Off-Standard*